







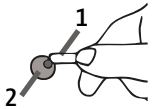

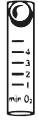







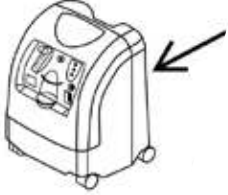
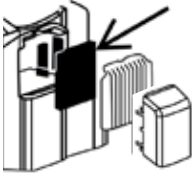





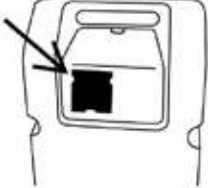



Instruction card Oxygen concentrator

<p>1 Always read the instructions.</p> 	<p>2 No open fire. Do not smoke. No e-smoker or vapes either. Do not use oil or greased based products.</p> 	<p>3 Ventilate the room well. For advice, see the ventilation guide supplied.</p> 
<p>4 For fire safety reasons, do not use electrical equipment that can generate sparks in the room where the concentrator is located and the client is present.</p> 	<p>5 Place the concentrator in a safe location, as directed by the service representative. The concentrator should always be used upright.</p> 	<p>6 Do not place the concentrator under a coat rack or against curtains. This will increase oxygen saturation.</p> 
<p>7 Extension cords may not be used.</p> 	<p>8 Technical maintenance may only be carried out by the service technician.</p> 	<p>9 Plug the concentrator into the wall socket and turn it on using the on/off button.</p> <p>aan <input checked="" type="radio"/></p> <p>uit <input type="radio"/></p>
<p>10 Slide the long oxygen hose (1) over the gray grommet (2). Check that the hose is securely attached.</p> 	<p>11 Firesafe. Place the firesafe between the nasal cannula and the long tube.</p> 	<p>12 Set the flow meter (number of liters per minute) as prescribed by your doctor.</p> 
<p>13 Not experiencing any flow? Check the flow by placing the nasal cannula and/or tube in a glass of water. If there are no bubbles, check the tube and the flow meter. If there is still no flow, contact Westfalen Medical.</p> 	<p>14 Turn off the concentrator after use. Turn off the concentrator when not in use.</p> <p>aan <input type="radio"/></p> <p>uit <input checked="" type="radio"/></p>	<p>15 Maximum length of connecting hose.</p> 
<p>16 We recommend replacing the nasal cannula once a week.</p> 	<p>17 The coarse filter should be cleaned weekly. See the back of this card for cleaning instructions.</p> 	<p>18 Malfunctions and/or questions: Tel. 0570-858 450</p>  <p>Westfalen Medical</p>



Instruction card Filters

Perfect 02	<p>1 Filters is located at the back.</p> 	<p>2 Vacuum the coarse filter weekly.</p> 
Perfect 02	<p>1 Filters are located on both sides.</p> 	<p>2 Vacuum coarse filters on both sides weekly.</p> 
Kröber 02 / 4.0	<p>1 Filters are located at the back.</p> 	<p>2 Vacuum the coarse filter weekly.</p> 
DeVilbiss 1025 / C525	<p>1 Filters are located at the back.</p> 	<p>2 Vacuum the coarse filter weekly.</p> 
Everflo	<p>1 Filters are located at the back.</p> 	<p>2 Vacuum the air intake weekly.</p> 