

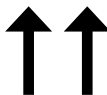


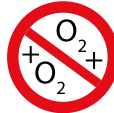

Instruction card Oxygen concentrator



westfalenmedical.com/kennisbank/

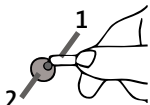

1 Always read the instructions. 	2 No open fire. Do not smoke. No e-smoker or vapes either. Do not use oil or greased based products. 	3 Ventilate the room well. For advice, see the ventilation guide supplied. 
4 For fire safety reasons, do not use electrical equipment that can generate sparks in the room where the concentrator is located and the client is present. 	5 Place the concentrator in a safe location, as directed by the service representative. The concentrator should always be used upright. 	6 Do not place the concentrator under a coat rack or against curtains. This will increase oxygen saturation. 
7 Extension cords may not be used. 	8 Technical maintenance may only be carried out by the service technician. 	9 Plug the concentrator into the wall socket and turn it on using the on/off button. aan ● uit ○
10 Slide the long oxygen hose (1) over the gray grommet (2). Check that the hose is securely attached. 	11 Firesafe. Place the firesafe between the nasal cannula and the long tube. 	12 Set the flow meter (number of liters per minute) as prescribed by your doctor. 
13 Not experiencing any flow? Check the flow by placing the nasal cannula and/or tube in a glass of water. If there are no bubbles, check the tube and the flow meter. If there is still no flow, contact Westfalen Medical.	14 Turn off the concentrator after use. Turn off the concentrator when not in use. aan ○ uit ●	15 Maximum length of connecting hose. 
16 We recommend replacing the nasal cannula once a week. 	17 The coarse filter should be cleaned weekly. See the back of this card for cleaning instructions. 	18 Malfunctions and/or questions: Tel. 0570 - 858 450  Westfalen Medical



Instruction card Filters

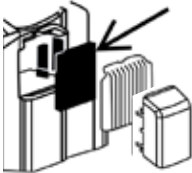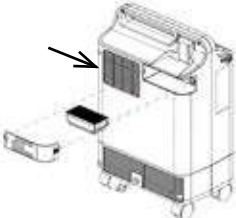Perfect 02	1 Filters is located at the back. 	2 Vacuum the coarse filter weekly. 
Perfect 02	1 Filters are located on both sides. 	2 Vacuum coarse filters on both sides weekly. 
Kröber 02 / 4.0	1 Filters are located at the back. 	2 Vacuum the coarse filter weekly. 
DeVilbiss 1025 / C525	1 Filters are located at the back. 	2 Vacuum the coarse filter weekly. 
Everflo	1 Filters are located at the back. 	2 Vacuum the air intake weekly. 
Philips Oxygenate 5	1 Filters are located at the side. 	2 Vacuum the coarse filter weekly. 