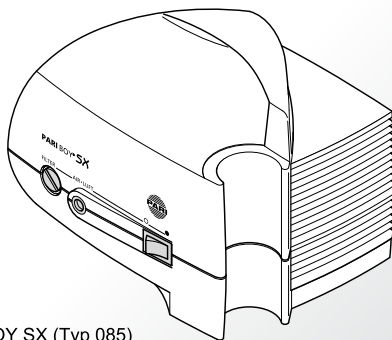
BOY SX JuniorBOY SX TurboBOY SX

BOY SX (Typ 085)

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en Instructions for use 27

Gebruiksaanweisung

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Instructions for use

Information as of: 2011-11. We reserve the right to make technical changes.

IMPORTANT INFORMATION	29
General	29
Structure of safety instructions	29
Using the compressor	29
Treatment of babies, children and anyone who requires assistance	31
PRODUCT DESCRIPTION.....	32
Components	32
Intended use	32
Working parts	33
Product variants and combinations	34
Operating life	34
PREPARATION FOR USE.....	35
Setup	35
Preparation.....	36
Performing the inhalation	37
Completing the inhalation.....	38
MAINTENANCE.....	39
Cleaning the compressor	39
Filter change	40
Servicing	41
Storage.....	41
TROUBLESHOOTING.....	42
TECHNICAL DATA.....	43
General	43
Classification according to DIN EN 60601-1	43
Information about electromagnetic compatibility	44
Recommended safety distances	44
Ambient conditions for operation.....	45

MISCELLANEOUS.....	46
Transportation and storage.....	46
Disposal.....	46
Terms and conditions of warranty	46
Contact	48
Explanation of symbols.....	48
SPARE PARTS.....	49

1 IMPORTANT INFORMATION

1.1 General

Please read these instructions for use and the instructions for use of the accessories supplied carefully and completely. Do not discard, so you can consult them at a later date. If you fail to comply with the instructions for use, injury or damage to the device cannot be ruled out.

1.2 Structure of safety instructions

Safety-critical warnings are categorised according to hazard levels in these instructions for use:

- The signal word **WARNING** is used to indicate hazards which, without precautionary measures, can result in serious injury or even death.
- The signal word **CAUTION** is used to indicate hazards which, without precautionary measures, can result in minor to moderate injury or impair treatment.
- The signal word **NOTE** is used to indicate general precautionary measures which are to be observed to avoid damaging the product during use.

1.3 Using the compressor

The compressor is an electrical device that is powered by supply voltage. It has been designed so that no live parts are accessible. However, if ambient conditions are unfavourable or if the compressor or power cord is damaged, this protection may no longer be assured. Therefore, please follow the instructions below to avoid damage to the device and the associated danger of contact with live parts (e.g., electric shock):

- The compressor is not splashproof. Protect the device and the power cord from liquid and moisture. Never handle the device, the power cord or the power plug with damp hands. In addition, the device must not be operated in a damp environment.

- Only use the PARI power cord to operate the compressor.
- Never leave the compressor unattended while it is in use.
- For safety reasons, always disconnect the power plug from the socket under the following circumstances:
 - if a malfunction occurs during operation
 - before cleaning and maintaining the device
 - immediately after use
- Always pull the power plug, not the cable, to disconnect the power cord from the socket.
- Make sure that the power cord is never kinked, pinched or jammed. Do not pull the power cord over sharp edges.
- Keep the compressor and the power cord away from hot surfaces (e.g., stove top, electric fire, open fire). Direct heat may damage the compressor housing or the power cord insulation.
- Keep the power cord away from domestic animals (e.g., rodents). They may damage the power cord insulation.
- The compressor must not be operated and/or the power plug must be unplugged from the socket immediately if the compressor or the power cord is damaged, or if a fault is suspected after the device has been dropped or suffered a similar shock.

1.4 Treatment of babies, children and anyone who requires assistance

Babies, children and anyone who requires assistance must be supervised constantly by an adult during inhalation therapy. This is the only way to ensure safe and effective treatment. Such persons often underestimate the hazards involved (e.g. strangulation with the power cord or the connection tubing), thus resulting in a risk of injury.

The product contains small parts. Small parts can block the respiratory tract and lead to a choking hazard. Therefore, make sure that you always keep the compressor, the nebuliser and the accessories out of the reach of babies and infants.

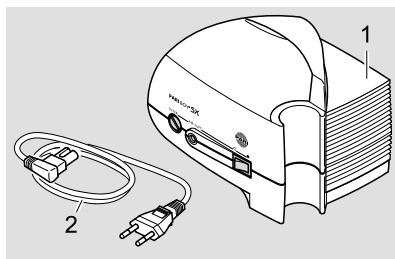
The compressor is an electrically powered device. Babies, infants, children, and anyone who requires assistance must be supervised constantly by an adult during inhalation therapy. Otherwise, safe and effective treatment cannot be guaranteed.

The device and the connection tubing at the compressor outlet can become very warm during normal operation. Therefore, particularly when dealing with individuals who require special assistance, ensure that their skin does not come into contact with the compressor. Prolonged contact may cause burns, particularly for patients with impaired temperature perception.

2 PRODUCT DESCRIPTION

2.1 Components

Check that all of the components belonging to your compressor are included in the package. If anything is missing, please notify the dealer from whom you purchased the PARI compressor immediately.

(1) Compressor

(2) Power cord

2.2 Intended use

The purpose of the compressor is to generate compressed air for operating a PARI nebuliser. The compressor must only be used for PARI nebulisers. It is intended exclusively for indoor use.

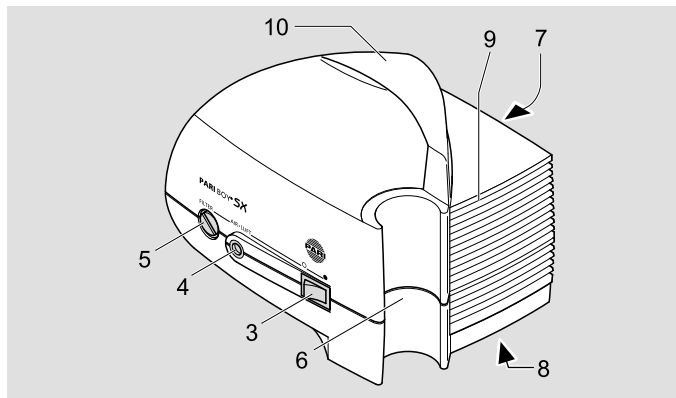
Any use for other purposes or use of nebulisers or accessories from other manufacturers may impair the quality of the treatment, cause damage to the device, or even result in injury.

PARI GmbH rejects any liability for damage caused by operating the compressor in a manner inconsistent with its intended use.

This particularly applies if the compressor is used in any way other than for its intended use or if nebulisers or accessories from other manufacturers are used. In such cases, all claims under the warranty shall also be void.

2.3 Working parts

The compressor includes the following working parts:

- (3) On/off switch
- (4) Air outlet
- (5) Filter holder
- (6) Holder for nebuliser
- (7) Power cord socket (rear of the device)
- (8) Type identification plate (bottom of device)
- (9) Ventilation slits
- (10) Carrying handle

2.4 Product variants and combinations

There are three different versions of the PARI BOY SX compressor:

- PARI JuniorBOY SX (compressor colour yellow)
including the LC SPRINT Junior nebuliser with the yellow nozzle attachment and soft baby mask with babybend
- PARI TurboBOY SX (compressor colour blue)
including the LC SPRINT nebuliser with the blue nozzle attachment and child mask
- PARI BOY SX (compressor colour blue)
including the LC SPRINT nebuliser with the blue nozzle attachment, child mask and PEP S system. A red nozzle attachment is also included in the supplied package

The compressor is suitable for use with the following nebulisers:

- All nebulisers in the PARI LC SPRINT family
- All nebulisers in the PARI LC family
- PARI LL nebulisers

Separate instructions for use accompany all nebulisers, whether they are included in the basic equipment or purchased subsequently.

2.5 Operating life

The compressor has an average expected operating life of:

- approx. 1,000 operating hours
- max. 5 to 6 years

3 PREPARATION FOR USE

3.1 Setup

The room in which the compressor is operated must satisfy certain conditions [See also: 6.5 Ambient conditions for operation]. The compressor is not intended for outdoor use.

CAUTION

When setting up the compressor, follow the instructions below to avoid any **risk of injury** or **damage to the device**:

- Place the compressor on a firm, flat, dust-free, dry surface. A soft base such as a sofa, a bed or tablecloth is not suitable because the compressor might fall.
- Route the power cord in such a way that it is not a tripping hazard and no one can become entangled in it.
- Make sure that children cannot pull on the power cord or connection tubing and cause the compressor to fall.
- Only operate the compressor in a low-dust environment. If it is operated in a very dusty environment (e.g., under the bed or in workshops), or on the floor, a larger amount of dust is likely to collect inside the housing, which may impair the quality of the treatment or damage the compressor.

WARNING

To avoid a **risk of fire** due to a **short circuit**, follow the instructions below when setting up the compressor:

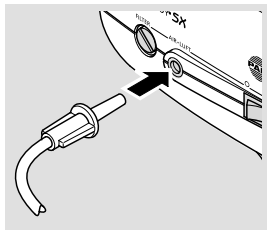
- Do not operate the compressor close to flammable objects such as curtains, tablecloths, or paper.
- Do not operate the compressor in areas where there is a risk of explosion or in the presence of oxidising gases (e.g., oxygen, nitrous oxide, flammable anaesthetics).

3.2 Preparation

Check the compressor housing, the power cord and the power plug for damage every time before using. Make sure that the local supply voltage complies with the voltage data marked on the compressor identification label. Only use the compressor if all components are intact. The compressor must not be operated if the device or the power cord is damaged, or if damage is suspected (e.g., after it has fallen or if there is a smell of burning plastic).

Prepare the device for an inhalation session as follows:

- Assemble the nebuliser as described in the instructions for use of the nebuliser.
- Insert the nebuliser in the holder on the compressor intended for this purpose.
- Insert the connection tubing of the PARI nebuliser with a slight twist into the air outlet on the compressor.

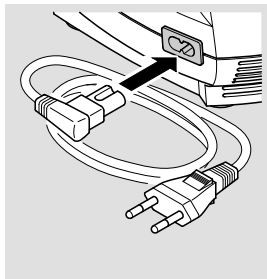
WARNING

Check carefully whether the other end of the PARI connection tubing connected to the compressor is connected to the PARI nebuliser.

When several tubing systems are present, there is a possible **danger of death** if different connection options are confused with each other. This applies particularly **in the case of patients in need of constant care** who must receive artificial nutrition or other infusion treatments.

- Connect the power cord to the power cord socket of the compressor.
- Fill the nebuliser as described in the instructions for use of the nebuliser.
- Plug the power plug into a suitable socket.

WARNING

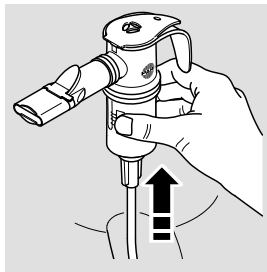
Liquids can conduct electricity, thereby posing a **risk of electric shock**. Therefore, never take hold of the power cord or the power plug with damp or wet hands.

3.3 Performing the inhalation

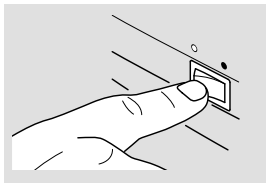
To avoid overheating the compressor, never place a cover over the device during operation, and ensure that the side ventilation slits are not obstructed. If the compressor is covered or if the ventilation slits are blocked, the compressor cannot be cooled properly.

Perform the inhalation as follows:

- Before starting the treatment, pull the nebuliser out of the holder.

- Start the compressor by pressing the on/off switch.
- Check that an aerosol is being generated before you start with the inhalation.

WARNING

In the event of **damage to the device** a person may come into contact with live parts, and there may be a **risk of electric shock**. Therefore, switch the compressor off and disconnect the power plug from the mains socket immediately if the compressor or the power cord is damaged or if damage is suspected (e.g., following a fall or if there is a smell of burning plastic).

- Perform the inhalation as described in the instructions for use of the nebuliser.

3.4 Completing the inhalation

- Switch the compressor off.
 - Place the nebuliser back in its holder.
 - Disconnect the power plug from the mains socket.
-  Complete disconnection from the mains is only certain when the power plug has been unplugged from the socket.

4 MAINTENANCE

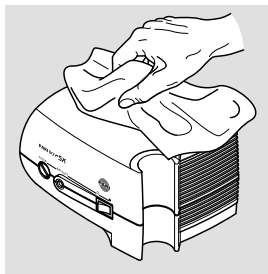
4.1 Cleaning the compressor

WARNING

Liquids can conduct electricity, thereby posing a **risk of electric shock**. Therefore, always switch the compressor off and disconnect the power plug from the mains socket before starting to clean it.

- Wipe the outer surface of the housing with a clean, damp cloth.

NOTICE

Liquids that get into the device can cause a **fault in the device**. Therefore, never spray any liquids onto the compressor or the power cord.

If liquid does get into the compressor, the device must not be used under any circumstances. Before using the compressor again, you must contact the PARI GmbH Technical Service department or the PARI Service Center [See also: 7.4 Contact]. In order to clean the nebuliser, the connection tubing and the accessories, follow the notes in the respective instructions for use.

4.2 Filter change

In normal operating conditions, the compressor filter must be replaced after every 200 operating hours, but at least once a year.

Also please check the filter at regular intervals (after every 10 inhalations). If it is soiled (grey or brown colour) or blocked, replace it. The filter must also be replaced with a new one if it has become damp.

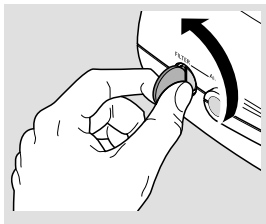
The filter cannot be cleaned and then reused!

NOTICE

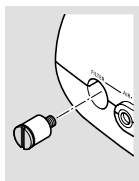
Only use air filters that are intended by PARI for use with your compressor, otherwise your **compressor might be damaged**.

To replace the filter, proceed as follows:

- Unscrew the filter holder from your compressor using a suitable coin.

- Pull the old filter out of the filter holder and fit the new filter in its place.
- Reinsert the filter holder together with the new filter in your compressor and screw it tight with the coin.

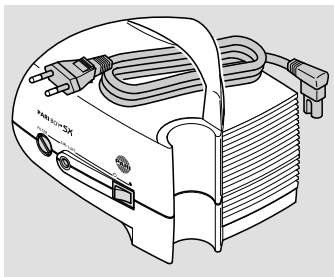
4.3 Servicing

The compressor must only be serviced by the PARI GmbH Technical Service department or a service center expressly authorised by PARI. If the compressor is opened or manipulated by anyone else, all claims under the warranty shall be void. In these cases, PARI GmbH will also accept no liability.

4.4 Storage

- Disconnect the power cord from the compressor.
- Wind up the power cord and stow it under the compressor handle.

NOTICE

The power cord can be damaged if it is kinked. Therefore, do not wind the power cord around the compressor.

Store the compressor and accessories safely out of continuous direct sunlight.

Always disconnect the compressor from the mains for storage because electrical devices that remain plugged present a potential hazard source.

5 TROUBLESHOOTING

Fault	Possible cause	Procedure
The compressor does not start up.	The power cord is not plugged into the device properly.	Check that the power cord is seated correctly in the compressor power cord socket.
	The power plug is not plugged into the mains socket properly.	Check that the power plug is seated correctly in the mains socket.
No medication is coming out of the nebuliser.	The nebuliser nozzle is blocked.	Clean the nebuliser.
	The connection tubing is not connected properly.	Check that both ends of the tubing are seated properly.
	The connection tubing is leaking.	Replace the connection tubing.

In the event of faults that are not listed in this chapter, or if the suggested procedure does not correct the fault, contact the PARI GmbH Service Center [See also: 7.4 Contact].

6 TECHNICAL DATA

6.1 General

Supply voltage [V] Mains frequency [Hz] Power consumption [A]	see back cover page
Housing dimensions [cm]	19.2 × 14.5 × 15 (W × H × D)
max. compressor flow (FreeFlow)	10.9 l/min.
Compressor flow ¹⁾	approx. 5.1 l/min.
Sound pressure level ²⁾	approx. 56 dBA
Pressure	approx. 1.6 bar
Weight	1.7 kg

1) towards PARI nebuliser nozzle (Ø 0.48 mm)

2) Maximum A-evaluated sound pressure level of the compressor
(in accordance with DIN EN 13544-1, section 26)

6.2 Classification according to DIN EN 60601-1

Type of electric shock protection	Protection class II
Degree of protection from electric shock of the part used (nebuliser)	Type BF
Degree of protection against water ingress in accordance with IEC 529 (IP rating)	No protection
Degree of protection when used in the presence of flammable mixtures of anaesthetics with air, with oxygen, or with nitrous oxide	No protection
Operating mode	Continuous operation

6.3 Information about electromagnetic compatibility

Electrical medical equipment is subject to special precautionary measures with regard to electromagnetic compatibility (EMC). Such equipment must only be installed and operated in accordance with EMC instructions.

Portable and mobile high-frequency communication devices can disrupt electrical medical equipment. Using accessories, converters and power cords other than those specified, with the exception of converters and power cords that the manufacturer of the medical electrical device sells as spare parts for internal components, can result in higher emission levels or lower the resistance to interference of the device.

The device must not be placed directly beside or on top of other devices for operation. If the medical electrical device must be placed beside or on top of other devices to operate it, it should be monitored constantly to ensure that it is operating properly in the arrangement used.

Technical data on electromagnetic compatibility (EMC instructions) in table format are included with the compressor. It can also be ordered from PARI GmbH or downloaded from the internet at www.pari.de/en/products under "Technical Data" on the respective product page.

6.4 Recommended safety distances

Recommended safety distances between portable and mobile high-frequency (HF) telecommunication devices and the PARI compressor:

The PARI compressor is intended for operation in an electromagnetic environment in which radiated HF transients are monitored. The user can help to prevent electromagnetic interference by observing the minimum safety distances between portable and mobile HF communication equipment

(transmitters) and the PARI compressor, as recommended below in accordance with the maximum output power of the communication device.

Safety distance depending on transmitting frequency:

Rated power of the transmitter [W]	150 kHz to 80 MHz $D = (3.5/3) \times \text{SQRT}(P)$ [m]	80 MHz to 800 MHz $D = (3.5/3) \times \text{SQRT}(P)$ [m]	800 MHz to 2.5 GHz $D = (7/3) \times \text{SQRT}(P)$ [m]
0.01	0.2	0.2	0.3
0.1	0.4	0.4	0.8
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters whose rated power does not appear in the table above, the safety distance can be calculated using the equation associated with the respective column, where "P" is the rated power of the transmitter in Watt (W) according to the information supplied by the device manufacturer.

6.5 Ambient conditions for operation

Ambient temperature	+10 °C to +30 °C
Relative humidity	30% to 75% (non-condensing)
Atmospheric pressure	700 hPa to 1,060 Pa

7 MISCELLANEOUS

7.1 Transportation and storage

Minimum ambient temperature	-25 °C (without monitoring of relative humidity)
Maximum ambient temperature	+70 °C (with relative humidity of up to 93%, non-condensing)
Humidity	0% – 93%
Atmospheric pressure	500 hPa – 1,060 hPa

7.2 Disposal

This product falls within the scope of the European Council Directive on Waste Electrical and Electronic Equipment (WEEE)¹⁾. Accordingly, this product must not be disposed of with domestic waste. The disposal regulations prevailing in the respective member countries must be observed (e.g., disposal by local authorities or dealers). Materials recycling helps to reduce the consumption of raw materials and protect the environment.

1) Directive 2002/96/EC of the EUROPEAN PARLIAMENT AND THE EUROPEAN COUNCIL of January 27, 2003 on waste electrical and electronic equipment.

7.3 Terms and conditions of warranty

PARI guarantees that your device, if used according to the instructions, will be free from defects in material and workmanship caused by the manufacturing process for the warranty period indicated on the warranty certificate beginning on the date of initial purchase. Claims under the warranty shall be subject to a limitation period of 12 months. The warranty provided by PARI applies in addition to the warranty obligation of your dealer. Your statutory rights with respect to your dealer in the event of defects

are not limited by the warranty or any claim under the warranty. The warranty certificate stamped by the dealer serves as your proof of warranty and ownership.

What does the warranty cover?

If, exceptionally, a defect is discovered, PARI will at its discretion repair or replace the device, or refund the purchase price of the product. If it is replaced, the replacement device may either be the same model or a model that is at least comparably equipped. Replacement or repair of the device shall not serve as the basis for a new warranty. All replaced old devices or parts shall become the property of PARI. Further claims are excluded. This applies particularly for any claims for compensatory damages. This disclaimer of warranty shall be ineffective in the event of injury to life, limb or health, in cases of wilful malfeasance and gross negligence, product liability and if substantive obligations under the warranty agreement are violated.

The warranty shall be cancelled if

- the device has been operated or used improperly with respect to the descriptions in the instructions for use
- damage is present that is attributable to the effects of water, fire, lightning, etc.
- the damage was caused by transporting the device incorrectly or a falling impact
- the device has been misused or not cared for correctly
- the serial number on the device has been changed, removed, or otherwise rendered illegible
- repairs, adaptations or modifications have been made to the device by persons not authorised by PARI

Moreover, the warranty does not cover wearing parts, that is to say device parts that are exposed to normal wear.

In the event of a complaint, please bring the entire device to your specialist dealer or send it to us packed in the original box, postage paid, together with the warranty certificate stamped by the dealer.

The "warranty period" begins on the date of purchase.

7.4 Contact

For all product information and in the event of defects or questions about usage, please contact our Service Center:

Tel.: +49 (0)8151-279 279 (German-speaking)
+49 (0)8151-279 220 (international)

For repairs and claims under the warranty, please send the device to the following address:

PARI GmbH – Technical Service
Holzhofstr. 10b, 82362 Weilheim, Germany

7.5 Explanation of symbols

The following symbols can be found on the compressor and/or the packaging:

	Follow the instructions for use (background = blue, symbol = white)
SN	Serial number of the device
	On/Off
	Alternating current
	Protection class of the part used: Type BF
	Protect device from moisture

	Protection class II device
	Minimum and maximum ambient temperature
	Minimum and maximum humidity
	Minimum and maximum air pressure
	The medical device was distributed commercially after 13 August 2005. This product must not be disposed of with normal domestic waste. The symbol of the refuse bin with a cross through it indicates that it must be collected separately.
	Manufacturer
	The compressor satisfies the basic requirements as set forth in Appendix I of Directive 93/42/EEC concerning medical devices.

8 SPARE PARTS

Description	Item no.
PARI BOY filters (5-pack)	041B4852
Filter screw with filter (only for compressor type 085)	085B1130
Power cord for PARI BOY SX compressors (1.5 m)	041E8236

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CERTIFICATE OF GUARANTEE

We grant a 4 year guarantee on the compressor,
commencing on the date of purchase.

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Confirmation of purchase: The appliance with the above appliance number was sold in its original packaging.

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©2012 PARI GmbH Spezialist für effektive Inhalation, 085D3002-A (3rd) 04/12