
**Kindermaske soft
Erwachsenenmaske soft**

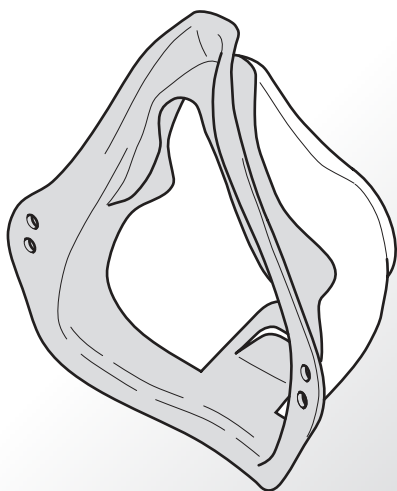CE 0123

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en Instructions for use

The PARI child mask soft and the PARI adult mask soft are accessories for PARI nebulisers (except the PARI TIA). They can be used for inhalation treatment of the lower respiratory tract if it is not possible to use a mouthpiece, or if the patient prefers to inhale through a mask. The PARI child mask soft is suitable for use by children aged about 4 and older.

- i** – In general, inhalation with the mouthpiece is the most effective way of treating the lower respiratory tract as this allows the greatest possible quantity of medication to reach the lung.
- Follow the instructions for assembly, inhalation, and cleaning the masks in the instructions for use of the respective nebuliser.
- The elastic strap on the masks cannot be disinfected or sterilised.
- **It is essential to ensure that the expiratory valve plate on your mask is pressed out before each use.**
- **The mask can only be sterilised with a special mask stabiliser. This is available from PARI as an accessory.**

Materials used: Polypropylene, thermoplastic elastomer

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©2010 PARI GmbH Spezialisten für effektive Inhalation, 041D0333-C-10/10